


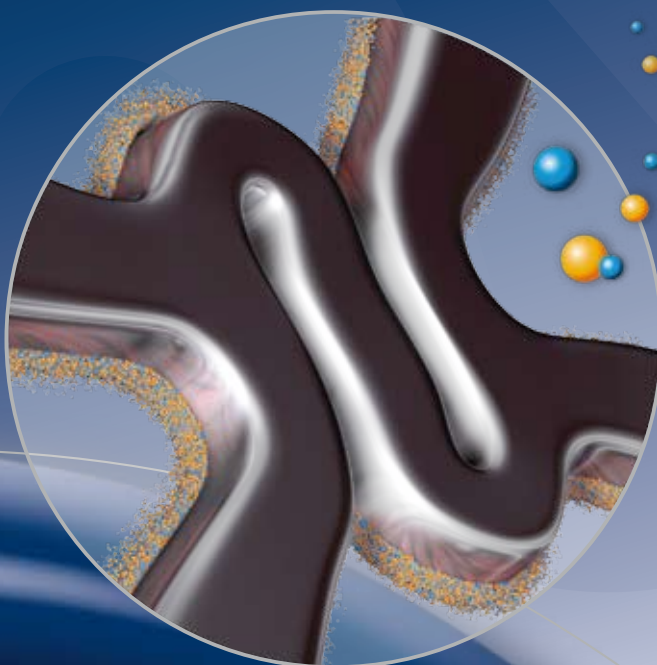

# BIOMATRIX

THE ABLUMINAL BIODEGRADABLE POLYMER DES *FLEX*™

From a DES to a BMS\*



BIOSENSORS  
INTERNATIONAL


## From a DES to a BMS\*

### Abluminal biodegradable coating



No drug carrier or drug inside the stent:

- Early BMS-like endothelial coverage<sup>1</sup>
- More targeted drug release
- Reduced systemic exposure

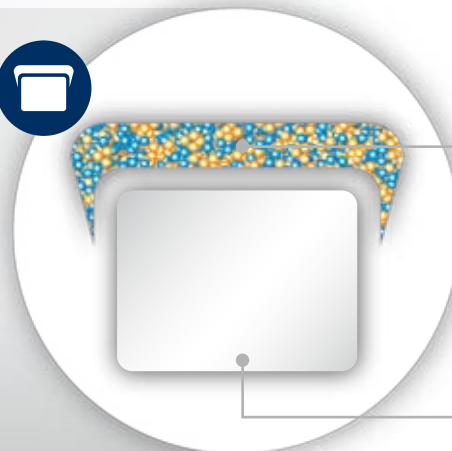

PLA biodegradation and BA9™ drug elution

from a DES



Abluminal coating absorbed after 6-9 months\*

to a BMS\*




# Proven safety of a DES with an abluminal biodegradable polymer




## Confirmed safety

- With a 16% reduction in MACE compared to the Cypher® Select™ stent at 2 years in LEADERS<sup>2</sup>
- With no late or very late stent thrombosis in STEALTH I out to 4 years<sup>4</sup>

### MACE (Cardiac Death, MI or TVR) at 2 years<sup>2</sup>

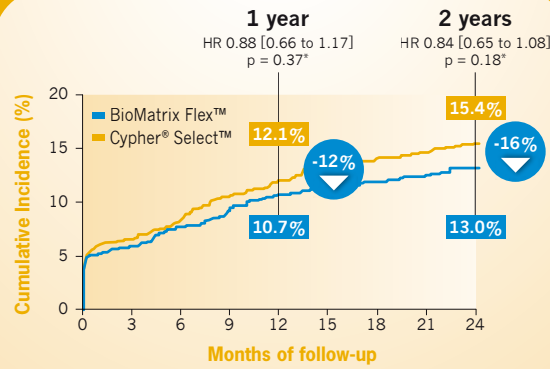

### Significant Reduction in Cardiac Mortality in Complex Patients at 2 Years<sup>3</sup> – Syntax Score High (>16)

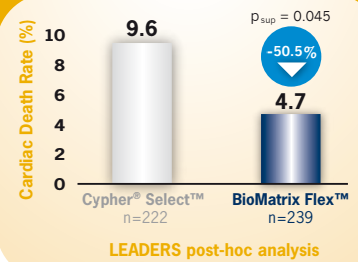

## Mortality benefit

The BioMatrix Flex™ stent reduces the cardiac death rate by 50.5% compared to the Cypher® Select™ stent at 2 years<sup>3</sup>

# Proven efficacy of the Biolimus A9™ drug




## Low Late Loss

- Lower level of late loss achieved in the LEADERS trial with the BioMatrix Flex™ stent at 9 months compared to the Cypher® Select™ stent<sup>8</sup>

In stent late loss in LEADERS = 0.13 mm<sup>8</sup>



### In-Stent Late Luminal Loss at 9 months<sup>9</sup>

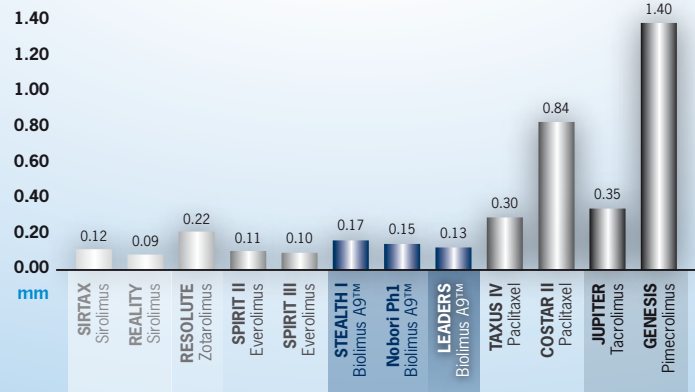


For descriptive purposes only – RCTs with different protocols should not be compared

# Enhanced deliverability\*\* with advanced stent design





## Improved flexibility<sup>10</sup>

The curved connectors combined with the Quadrature Link™ design give the Juno™ stent platform improved flexibility, while preserving stent security and vessel scaffolding

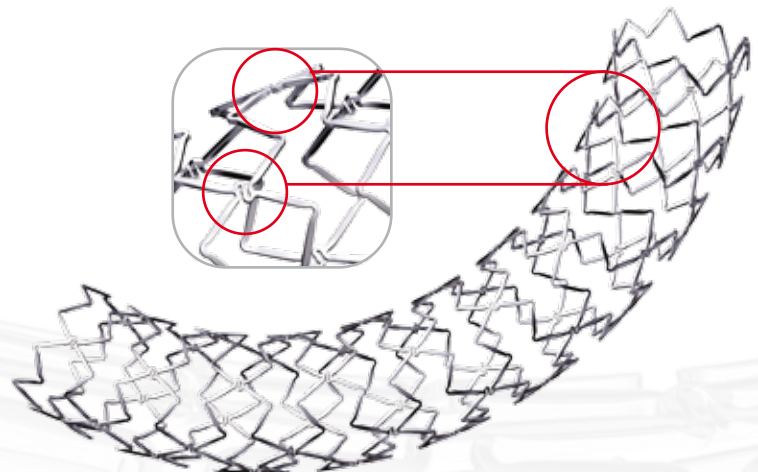

\*\* Deliverability defined as a combination of flexibility, trackability and pushability

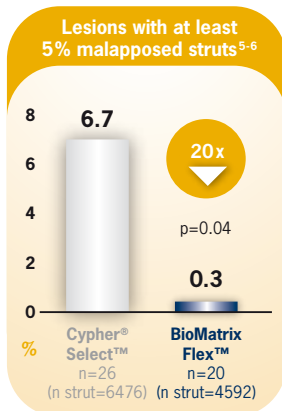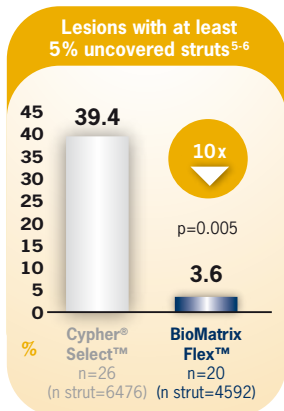


## Superior strut coverage

10 times better strut coverage of the BioMatrix Flex<sup>™</sup> stent than the Cypher<sup>®</sup> Select<sup>™</sup> stent at 9 months<sup>5-6</sup>


- The BioMatrix Flex<sup>™</sup> stent achieves 96.4 % of near to complete strut coverage

## Superior stent apposition

More than 20 times better stent apposition of the BioMatrix Flex<sup>™</sup> stent than the Cypher<sup>®</sup> Select<sup>™</sup> stent at 9 months<sup>5-6</sup>

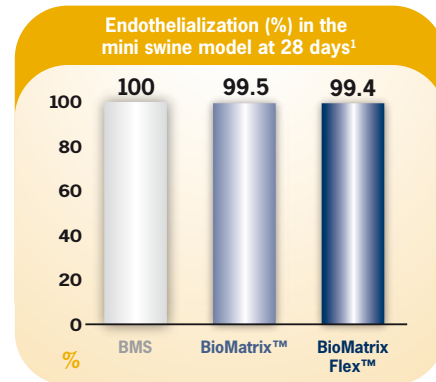


In the animal model, the BioMatrix Flex<sup>™</sup> stent achieves a similar level of re-endothelialization compared to its bare metal stent platform

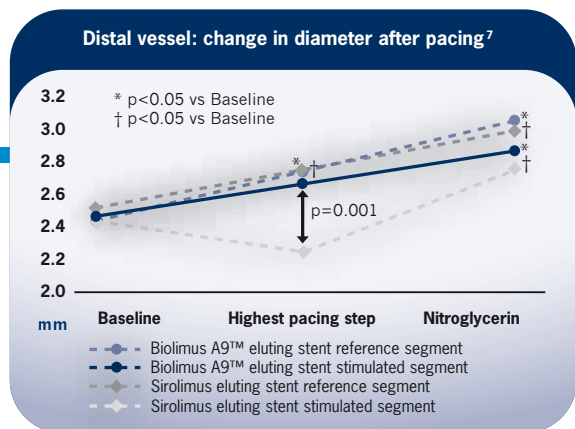

## BA9<sup>™</sup> drug / Biodegradable PLA technology demonstrates preserved vasomotion<sup>7</sup>


- Abnormal vasomotion (vasoconstriction) was observed for the sirolimus eluting stent group after high pacing stimulation, while normal vasomotion (vasodilatation) was observed in the BA9<sup>™</sup> eluting stent group

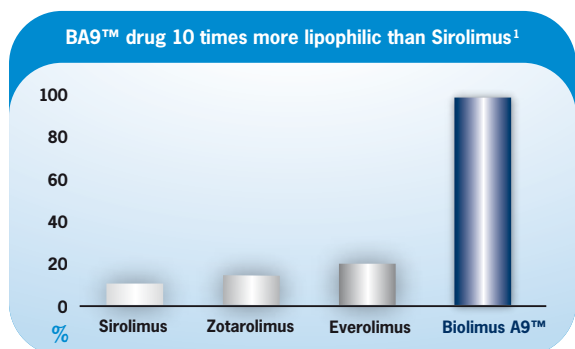

## Highest lipophilicity of the common limus drugs<sup>1</sup>


- Minimizes systemic exposure and reduces the drug circulating in the bloodstream
- Due to high lipophilicity, the drug is rapidly absorbed by tissue

## Improved flexibility<sup>10</sup>

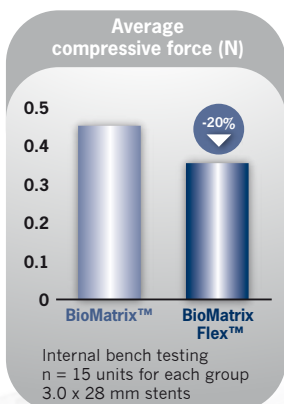


Less force needed to bend the BioMatrix Flex<sup>™</sup> stent for improved deliverability\*\*

## Improved trackability<sup>10</sup>

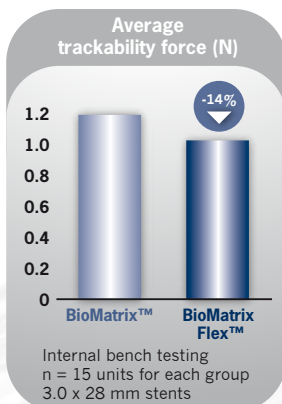

Less force needed with the BioMatrix Flex<sup>™</sup> stent to navigate through tortuous vessels for improved deliverability\*\*

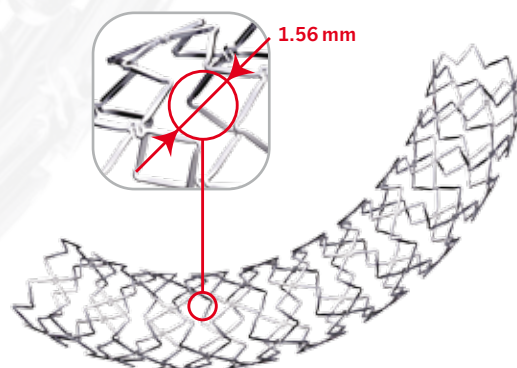

## Larger side branch access<sup>10</sup>

- Improved initial cell opening to facilitate the side branch access

## Taking the LEAD in DES Clinical Excellence

Our trials gather clinical data for this technology from a wide range of patients, including those with single de novo lesions, multiple vessel disease, acute coronary syndromes, bifurcations, left main, small vessels and extreme long lesions.

Trial	n	Planned FU	Status	Comparator
STEALTH PK	27	1 yr	Completed	Single Arm
STEALTH I	120	5 yrs	5 yrs FU completed	Gazelle™ (BMS)
BEACON I	292	1 yr	Completed	Single Arm
LEADERS	1707	5 yrs	2 yrs FU completed	Cypher® Select™ (DES)
BEACON II	497	5 yrs	30 d FU available	Single Arm
e-BioMatrix	PMS 1122 PMR 4000	5 yrs	PMS Enrollment Completed PMR Currently enrolling	Single Arm
e-BioMatrix India	PMD 1000 PMR 4000	5 yrs	Currently enrolling	Single Arm

From "Single de Novo"

### STEALTH PK n=27

Single Arm Registry, Pharmacokinetics

Primary Endpoint: Biolimus A9™ drug concentration at 30 days and 6 months

### STEALTH I n=120

Randomized Multicenter Clinical Trial – BioMatrix™ vs Gazelle™ (2:1)

Primary Endpoint: In-Stent Late Loss at 6 months

Results: 0.09 vs 0.48 mm ( $p < 0.01$ )

### BEACON I n=292

Multicenter Single Arm Registry

Primary Endpoint: TVR at 6 months

Results: 2.1 %

### LEADERS n=1707

Randomized Multicenter All-Comers Clinical Trial

BioMatrix Flex™ vs Cypher® Select™

Primary Endpoint: CV death, MI, clinically-indicated

TVR at 9 months

Results: ●

### BEACON II n=497

Multicenter Single Arm Post Market Registry

Primary Endpoint: MACE at 12 months

MACE defined as Cardiac Death, Ischemic Driven MI

(Q-wave and NQ-wave) and Ischemic Driven TLR (PTCA and CABG)

### e-BioMatrix n=+5000

Multicenter Single Arm Post Market Registry

PMS: n=1122

PMR: n=4000

Primary Endpoint: MACE at 12 months

MACE defined as Cardiac Death,

MI (Q-wave and NQ-wave), or justified TVR

### e-BioMatrix India n=5000

Multicenter Single Arm Post Market Registry

PMD: n=1000


PMR: n=4000

Primary Endpoint: MACE at 12 months

MACE defined as Cardiac Death,

MI (Q-wave and NQ-wave), or justified TVR


### Primary Endpoint Cardiac Death, MI or TVR at 9 months<sup>a</sup>

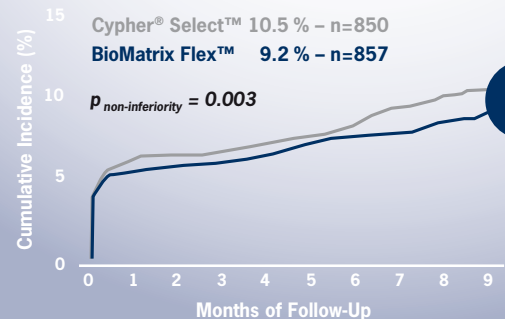


PMS = Post Market Surveillance

PMR = Post Market Registry

PMD = Post Market registry Diabetics

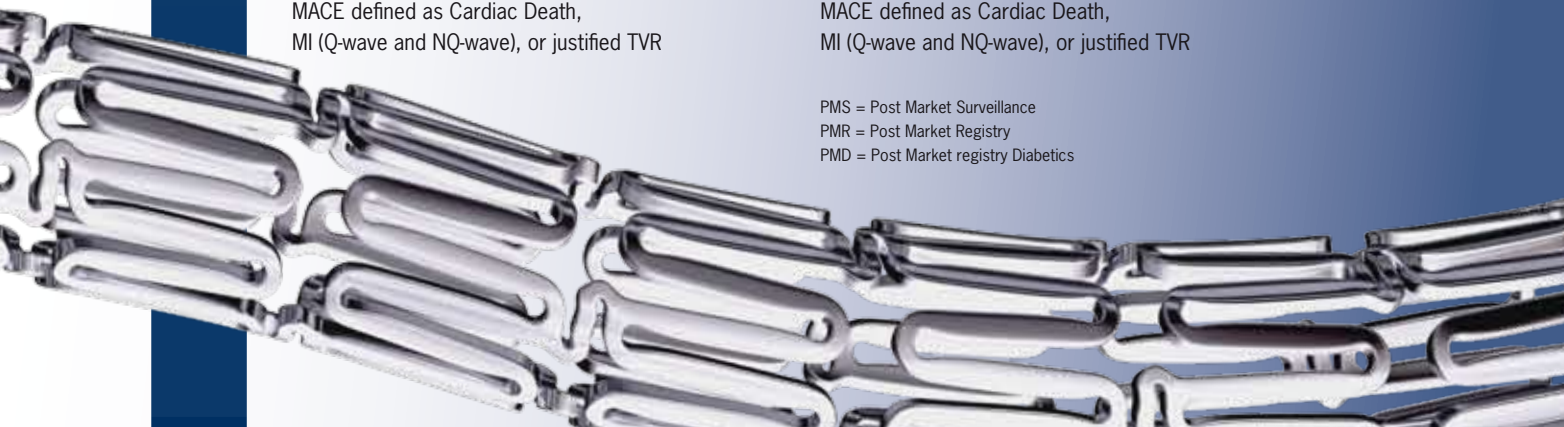

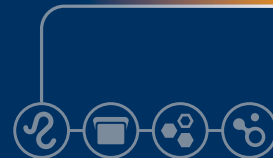

## Ordering Information

	Stent Length (mm)					
Stent Diameter (mm)	8	11	14	18	24	28
2.25	BMX-2208	BMX-2211	BMX-2214	BMX-2218	BMX-2224	BMX-2228
2.50	BMX-2508	BMX-2511	BMX-2514	BMX-2518	BMX-2524	BMX-2528
2.75	BMX-2708	BMX-2711	BMX-2714	BMX-2718	BMX-2724	BMX-2728
3.00	BMX-3008	BMX-3011	BMX-3014	BMX-3018	BMX-3024	BMX-3028
3.50	BMX-3508	BMX-3511	BMX-3514	BMX-3518	BMX-3524	BMX-3528
4.00	BMX-4008	BMX-4011	BMX-4014	BMX-4018	BMX-4024	BMX-4028

Biosensors International Group, Ltd. licenses its proprietary BA9™ drug and PLA technology to Terumo Corporation (Nobori®), Devax, Inc. (AXXESS™) and Xtent, Inc. (XTENT®).



\* In vivo testing in porcine model demonstrates abluminal coating is absorbed after 6 to 9 months – Data on file at Biosensors Intl

1. Data on file at Biosensors Intl
2. Klauss, V., LEADERS, 2-year Clinical Follow-Up, oral presentation, TCT 2009
3. Wykrzykowska, J., oral presentation, Late Breaking Trial Session, EuroPCR 2010
4. Grube, E., Safety and Performance Evaluation of Biosensors Biolimus A9™ Eluting Stent (BioMatrix™) STEALTH I: a 4-year safety follow-up, e-poster, TCT 2008 – The BioMatrix™ stent was used in the STEALTH I clinical trial
5. Di Mario, C., OCT: Differences between biodegradable and durable polymers – Insights from the LEADERS trial, oral presentation, TCT 2008
6. Barlis, P. et al., An optical coherence tomography study of a biodegradable vs. durable polymer-coated limus-eluting stent: a LEADERS trial sub-study; European Heart Journal (2010) 31, 165-176
7. Hamilos, M. et al. on behalf of the Nobori Core investigators, Differential Effects of Drug-Eluting Stents on Local Endothelium-Dependent Coronary Vasomotion, J Am Coll Cardiol 2008 51:2123-2129
8. Windecker, S. et al., Biolimus-eluting stent with biodegradable polymer versus Sirolimus-eluting stent with durable polymer for coronary revascularization (LEADERS): a randomised non inferiority trial; The Lancet 2008; 372 No. 9644: 1163-1173
9. Windecker, S. et al., NEJM 2005; Morice, M.-C. et al., JAMA 2006; Serruys, P.W. et al., EuroIntervention 2006; Stone, G.W. et al., JAMA 2008; Costa, M. et al., Am J Card 2006; Chevalier, B. et al., EuroIntervention 2006; Ostojic, M. et al., EuroIntervention 2008; Stone, G.W. et al., NEJM 2004; Krukoff, M.W., JACC 2008; Verhey, S. et al., ACC/SCAI 2008
10. Compared to the BioMatrix™ stent platform – Internal bench testing – Flexibility and trackability: n=15 in each group, 3.0x28mm stents – Side branch access: n=2 in each group, 3.0x28mm/3.0x18mm stents – Bench test results may not necessarily be indicative of clinical outcomes

BioMatrix Flex™ Drug Eluting Stent System is CE approved.

**CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device.

BioMatrix Flex, BioMatrix, Juno, Biolimus A9 and BA9 are trademarks or registered trademarks of Biosensors International Group, Ltd. in the United States and other countries.

All cited trademarks are the property of their respective owners.

Not available for sale in the United States and certain other countries.

© 2010 Biosensors International Group, Ltd. All rights reserved.

[www.biosensors.com](http://www.biosensors.com)



### BIOSENSORS EUROPE SA

Rue de Lausanne 29  
1110 Morges  
Switzerland  
Tel: +41 (0)21 804 80 00  
Fax: +41 (0)21 804 80 01

### BIOSENSORS INTERVENTIONAL TECHNOLOGIES PTE LTD

Blk 10 Kaki Bukit Avenue 1  
#06-01/04 - Kampong Ubi Industrial Estate  
Singapore 417942  
Tel: +65 6213 5725  
Fax: +65 6213 5737